

Take control.



LithAssist™

SUCTION CONTROL FOR LASER LITHOTRIPSY



www.cookmedical.com

LithAssist™

SUCTION CONTROL FOR LASER LITHOTRIPSY

The LithAssist Suction Control for Laser Lithotripsy is a lightweight, handheld device. It gives you one-handed control of suction during lithotripsy of kidney or bladder stones. Add LithAssist to your procedures and eliminate the need for an assistant who is dedicated to controlling suction.


Use the LithAssist in conjunction with a laser such as the H-30™ Holmium Laser System in order to achieve increased control over all aspects of the procedure. When using the laser to break up the stone, simultaneously suction the fragments and fluids using the LithAssist. This process may ultimately reduce your time in the OR.





Benefits

- It gives you single-handed control of the suction. This allows you to suction fragments and fluids simultaneously during laser lithotripsy without the help of an assistant.
- It works with your existing holmium laser system. This eliminates the need to make an additional capital investment for PCNL.
- It locks the laser fiber in place. This allows you to maneuver the tip of the fiber with precision and control.
- It avoids the need to make direct contact with the stone during the procedure. This may reduce trauma to the urothelial tissue.

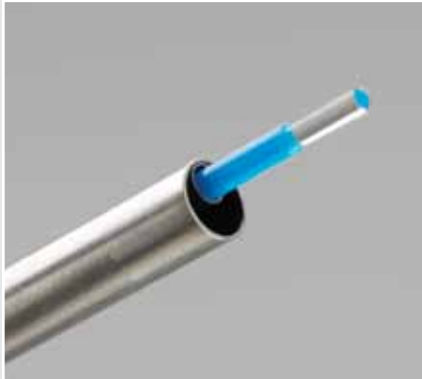

LithAssist™

SUCTION CONTROL FOR LASER LITHOTRIPSY

The LithAssist has a rigid 11.6 Fr dual-lumen cannula that is attached to an ergonomic handle for control. The smaller 5 Fr lumen provides access for the introduction of a laser fiber for use during lithotripsy of kidney or bladder stones. The larger lumen facilitates the suction of fluid, soft tissue, and stone fragments. The LithAssist features a trigger mechanism that enables you to manually control the suction strength.

Attach the tubing from an independent vacuum source to the universal adapter at the bottom of the LithAssist.

Order Number	Reference Part Number	Cannula Length cm	Cannula OD Fr	Inner Lumen ID Fr
G23810	LA-116038	38	11.6	5

Some products or part numbers may not be available in all markets. Contact your local Cook representative or Customer Service for details.

Customer Service

EMEA: EDI - www.cookmedical.com/edi.do
 Distributors: +353 61239240, ssc.distributors@cookmedical.com
 Austria: +43 179567121, oe.orders@cookmedical.com
 Belgium: +32 27001633, be.orders@cookmedical.com
 Denmark: +45 38487607, da.orders@cookmedical.com
 Finland: +358 972519996, fi.orders@cookmedical.com
 France: +33 171230269, fr.orders@cookmedical.com
 Germany: +49 6950072804, de.orders@cookmedical.com
 Hungary: +36 17779199, hu.orders@cookmedical.com
 Ireland: +353 61239252, ie.orders@cookmedical.com
 Italy: +39 0269682853, it.orders@cookmedical.com
 Netherlands: +31 202013367, nl.orders@cookmedical.com
 Norway: +47 23162968, no.orders@cookmedical.com
 Spain: +34 912702691, es.orders@cookmedical.com
 Sweden: +46 858769468, se.orders@cookmedical.com
 Switzerland - French: +41 448009609, fr.orders@cookmedical.com
 Switzerland - Italian: +41 448009609, it.orders@cookmedical.com
 Switzerland - German: +41 448009609, de.orders@cookmedical.com
 United Kingdom: +44 2073654183, uk.orders@cookmedical.com

www.cookmedical.com

Americas: EDI - www.cookmedical.com/edi.do
 Phone: +1 812.339.2235, 800.457.4500, Fax: 800.554.8335
 E-mail: orders@cookmedical.com

Australia:
 Phone: +61 738411188, 1800777222, Fax: +61 738411288, 1800077283
 E-mail: cau.custserv@cookmedical.com

